



# Enhanced acute dialysis care

DuoGlide™ Short-term Dual Lumen Dialysis Catheter



# DuoGlide™

## Short-term Dialysis Catheter

### Optimal performance<sup>+</sup>

- Bench test data demonstrated flow rates of 400mL/min with pressures of less than 250 mmHg.
- Designed for kink resistance
- Recirculation less than 2% on average during normal flows or when the arterial and venous lumens are reversed

### Flexibility and utility for you and your patient


- DuoGlide™ Dual Lumen Catheters are indicated for use in attaining short-term (less than 30 days) vascular access for hemodialysis, hemoperfusion and apheresis therapy via the jugular, subclavian or femoral vein.
- 30 cm length option for femoral placements
- Straight or curved extension legs

### Safety and compliance

- Safety components address safety concerns for users and facilitate compliance with federal regulations

### Chlorhexidine gluconate

- ChlorPrep™ One-step Applicator for preparation and maintenance of catheters
- Do not use on patients with known allergies to chlorhexidine gluconate or isopropyl alcohol

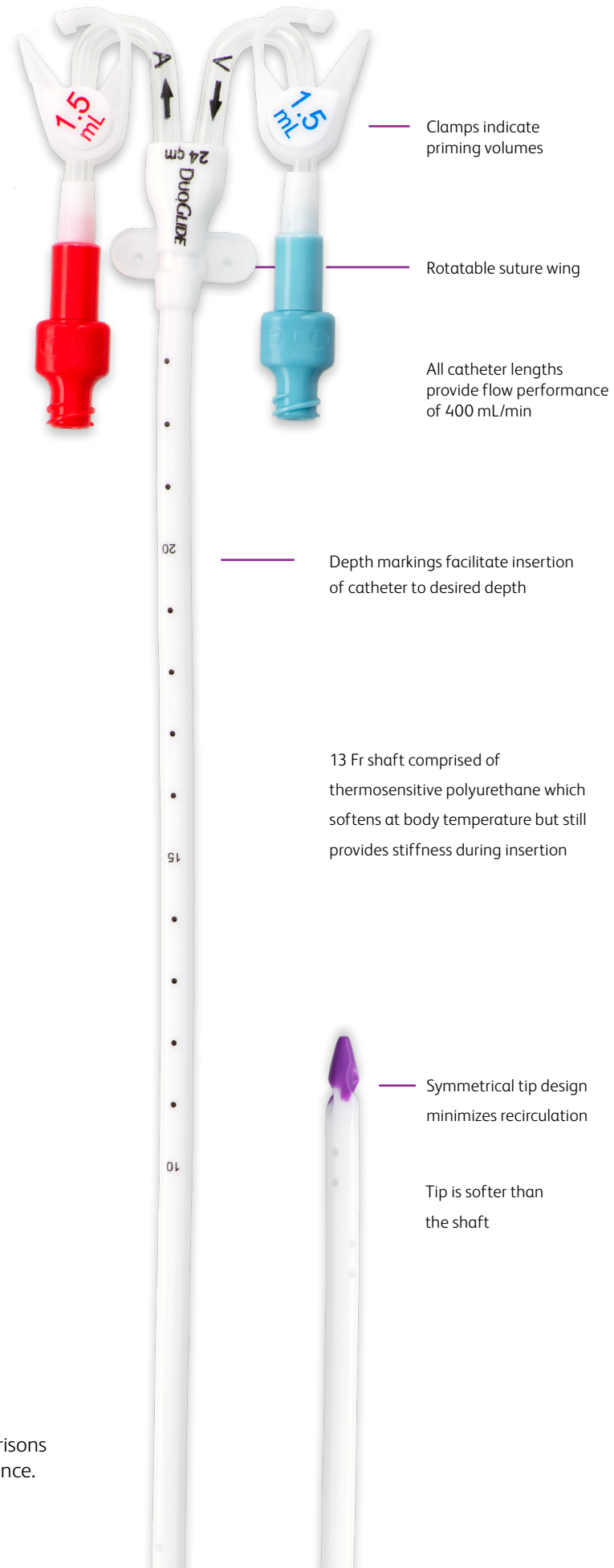

Clamps indicate priming volumes

Rotatable suture wing


All catheter lengths provide flow performance of 400 mL/min

Depth markings facilitate insertion of catheter to desired depth

13 Fr shaft comprised of thermosensitive polyurethane which softens at body temperature but still provides stiffness during insertion


Symmetrical tip design minimizes recirculation

Tip is softer than the shaft

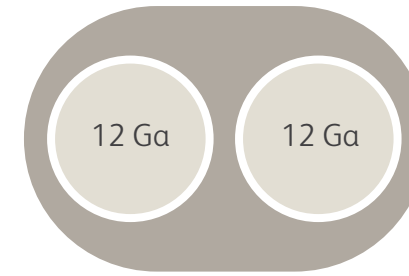

### Flow performance<sup>+</sup>

- Flow rates of 400 mL/min with pressures of less than 250 mmHg
- Double barrel lumen design permits higher flows than the competitive design

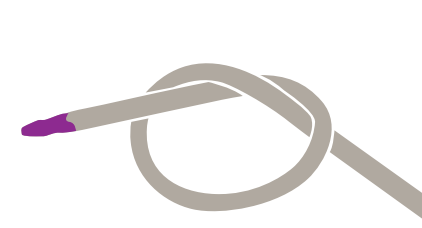

13 Fr DuoGlide™ Catheter




13 Fr Dual Lumen Mahurkar™

### Kink-resistance

- 13 Fr oval catheter shaft designed for kink resistance
- The DuoGlide™ Dialysis Catheter kinks at 1/2 the diameter of the Mahurkar™ Dual Lumen Dialysis Catheter\*

(Note: The product images shown are for illustrative purposes only and not an accurate description of kinking properties of either device.)

13 Fr DuoGlide™ Dialysis Catheter kink diameter =  
11.5 Fr Mahurkar™ Dual Lumen Dialysis Catheter kink diameter =

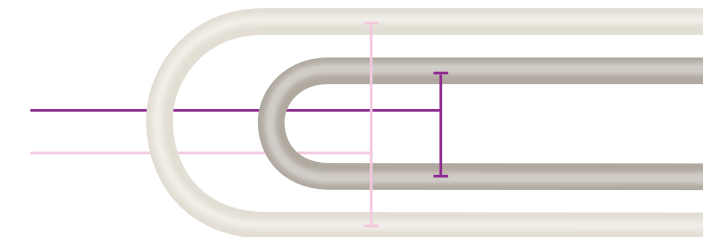

<sup>+</sup> Bard Access Systems Verification Testing Competitive Comparisons M548-034-005, 2016. This is not indicative of clinical performance.

<sup>+</sup> Bench test data on file. This is not indicative of clinical performance.

## Standard Insertion Kit

Product code	Catheter insertion length (cm)	Extension legs
5623150	15	Straight
5623200	20	Straight
5623240	24	Straight
5623300	30	Straight
5624120	12.5	Curved
5624150	15	Curved
5624200	20	Curved
5624240	24	Curved

## DuoGlide™ Short-term Dialysis Standard Insertion Kit contents

1	DuoGlide™ Short-term Dialysis Catheter	1	End cap
1	Attachable suture wing	1	StruXure™ Guidewire 0.035 in x 70 cm
1	Dualator® Vessel Dilator 11-13 Fr	1	Introducer needle, 18 Ga x 2.75 in
1	Dualator® Vessel Dilator 12-14 Fr	1	Transparent dressing

BD-Canada, Mississauga, ON L5N 0B3  
1.800.387.9473

[bd.com](http://bd.com)

BD, the BD Logo, Bard, Chloraprep, DuoGlide, Dualator, and StruXure are trademarks of Becton, Dickinson and Company. All other trademarks are the property of their respective owners. ©2023 BD and its subsidiaries. All rights reserved. BD-80941

