

BD[®] Nexiva[™]

Closed IV Catheter System—Dual Port

Catheter


- + High-pressure extension set*
- + Stabilization platform
- + Access port 1
- + Access port 2



This document is not intended as a supplement to or replacement for the Instructions for Use. Please consult product labels and inserts for instructions for use, indications, contraindications, warnings and precautions.


* 24 gauge catheter systems should not be used with power injectors.



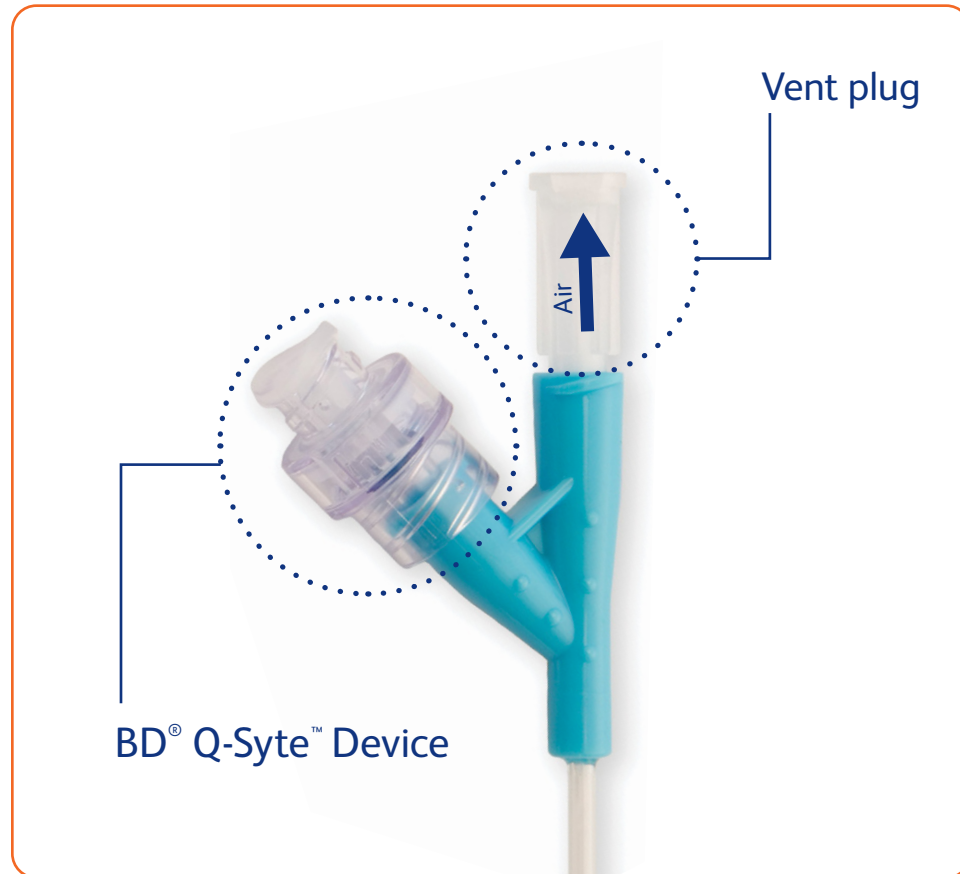


Secure connections

- Vent plug must remain in place during insertion for
 - Flashback
 - Blood containment
- BD Q-Syte™ luer access split septum

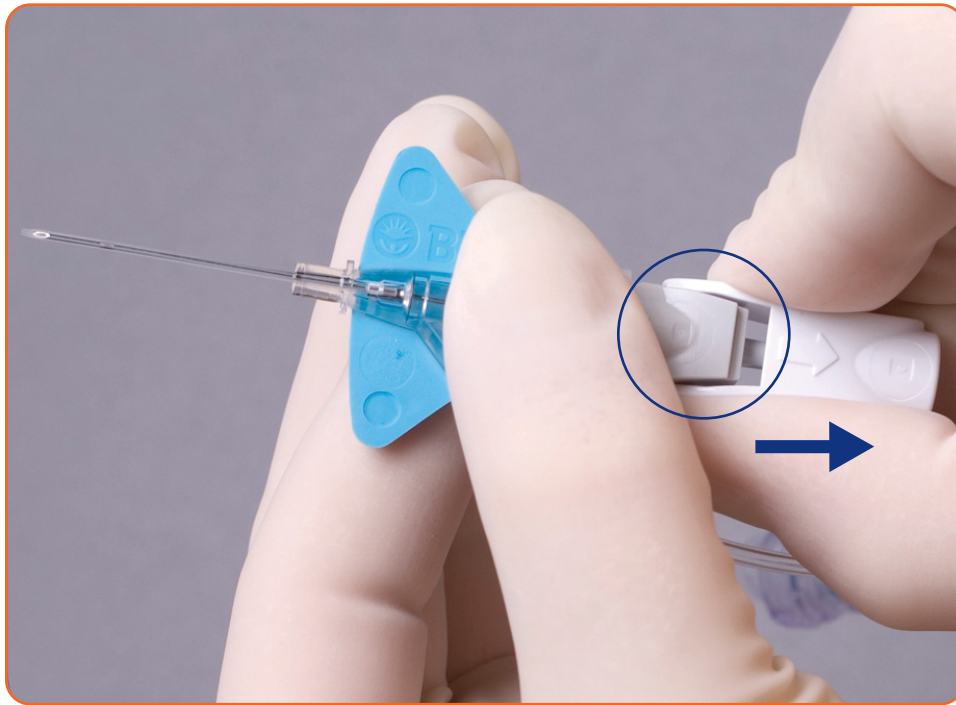

Release seal

- Loosens catheter for easier advancement
- Pull back ~ 1/8 inch

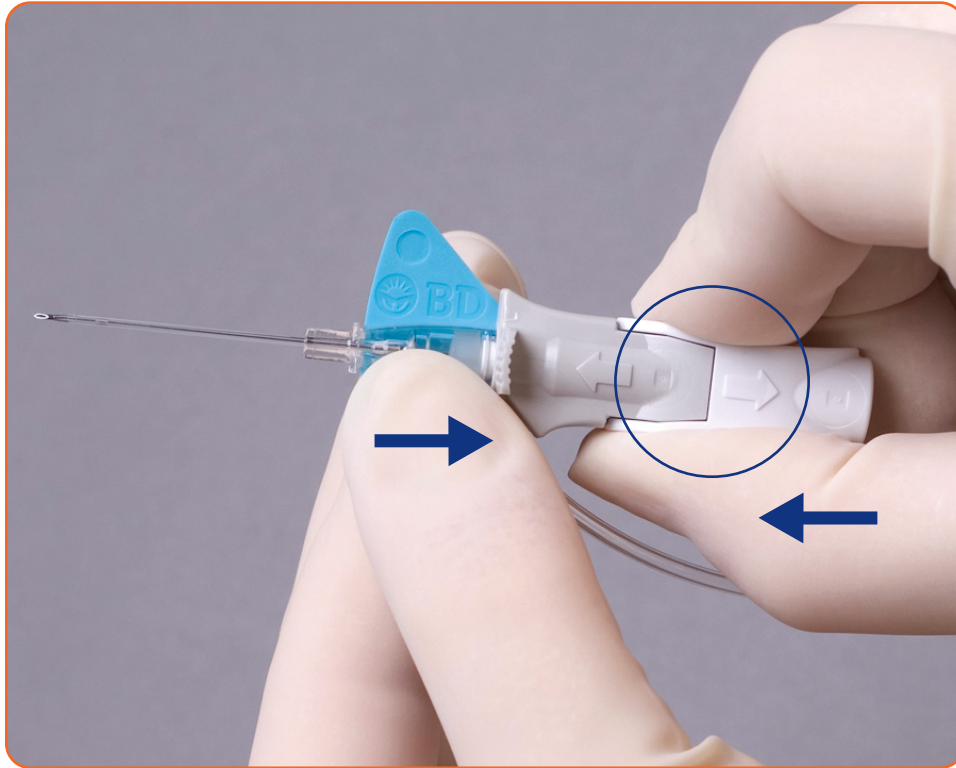

“Color to white
everything tight”


- Push colored platform and white grip back together


Keep it sharp



BD® Instaflash™ Needle Technology

- Immediate blood return is seen along the catheter

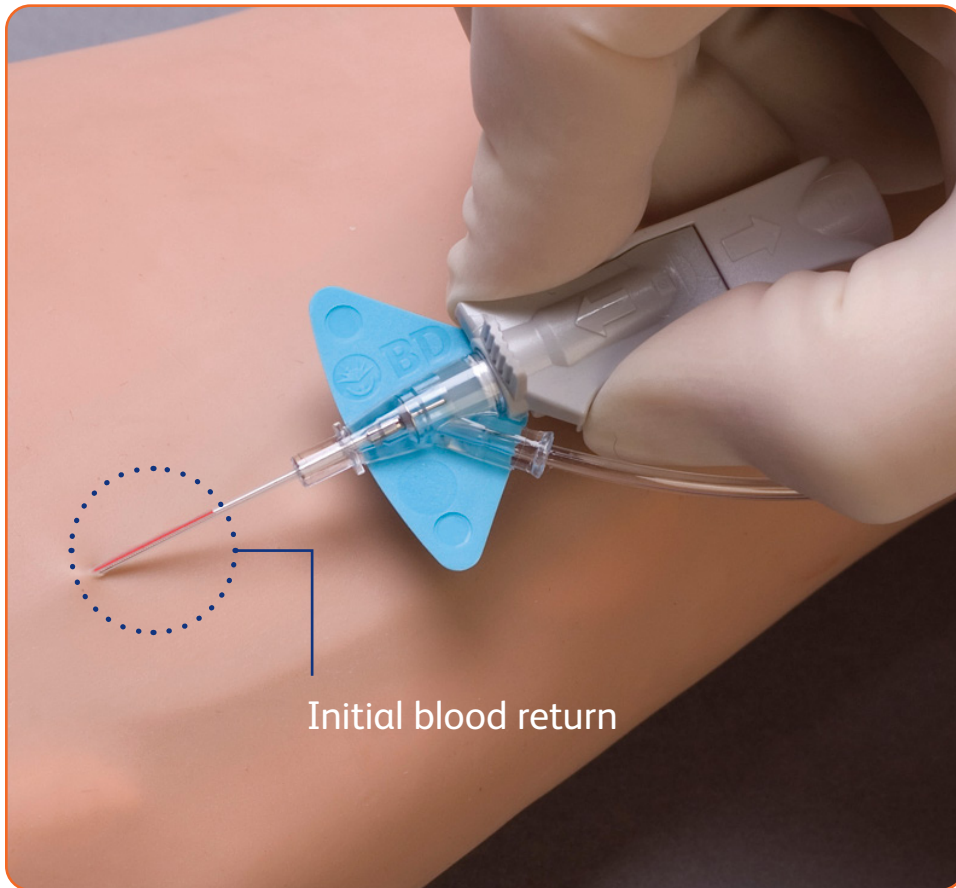


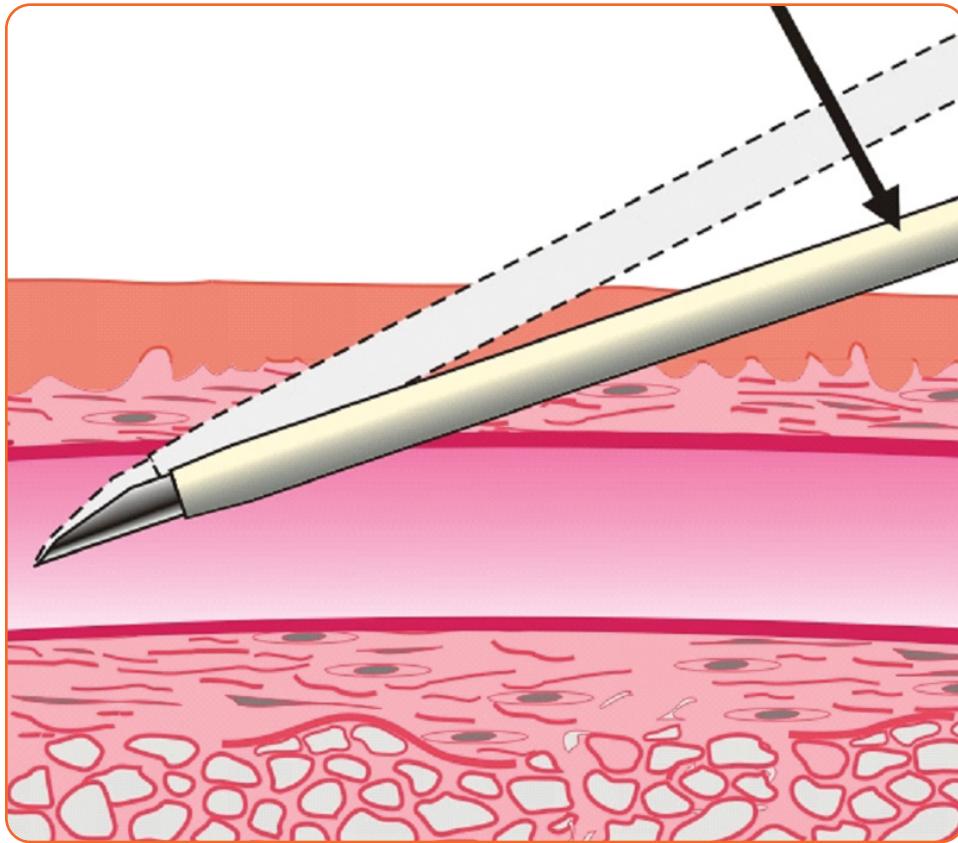



Lower and advance

- Lower and advance the entire catheter and needle unit slightly to ensure that the catheter tip is within the vessel


Forward first

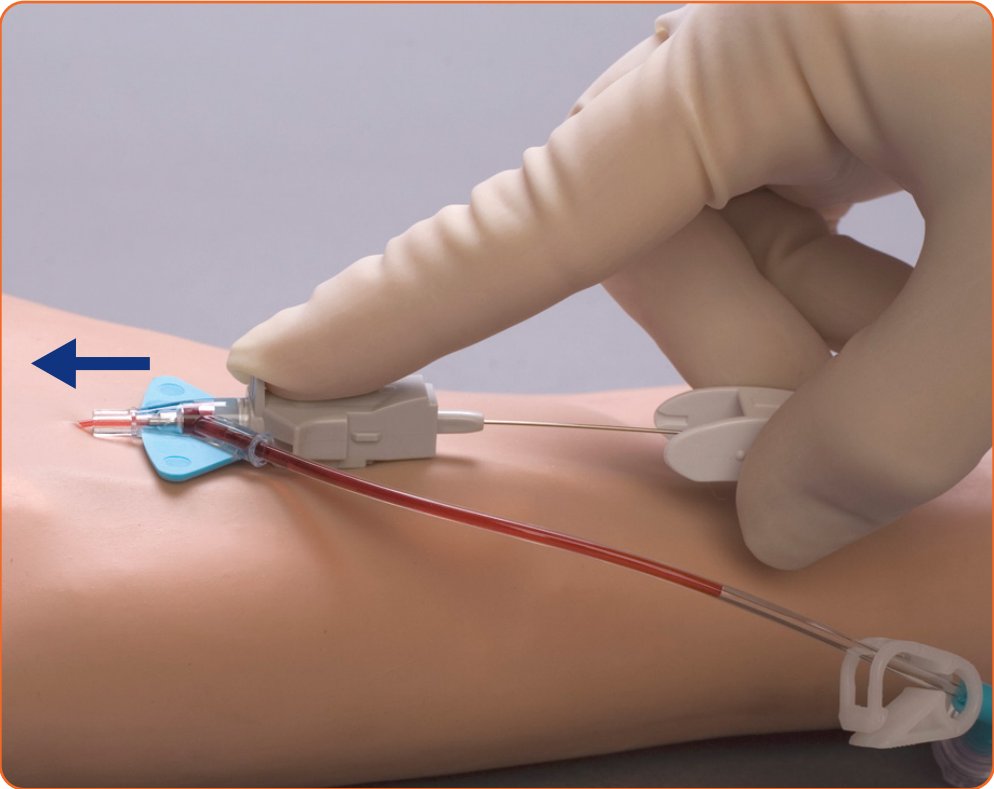


Pad to tab

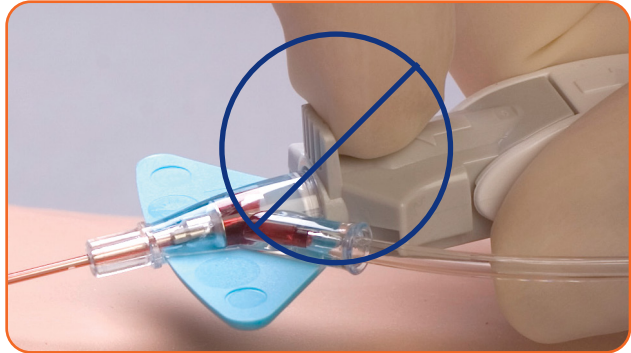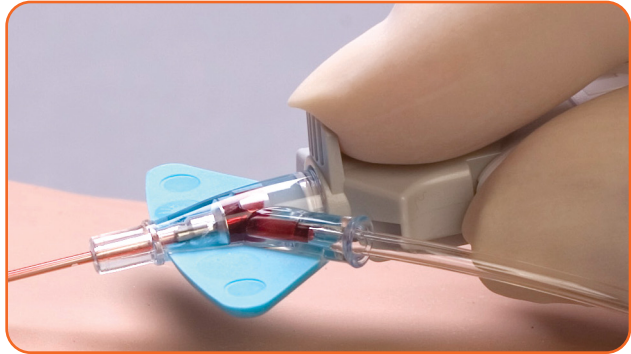

Forward first


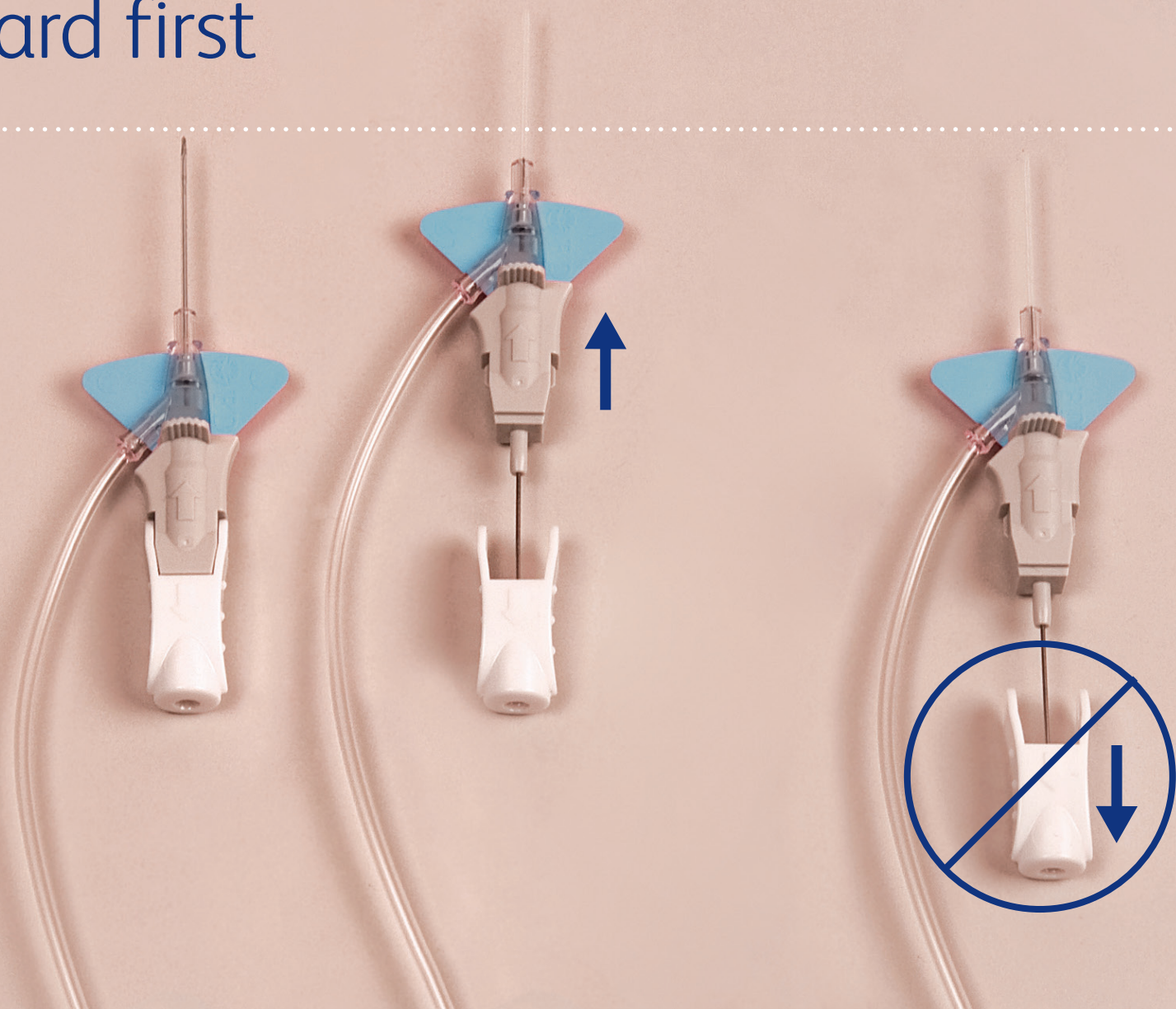


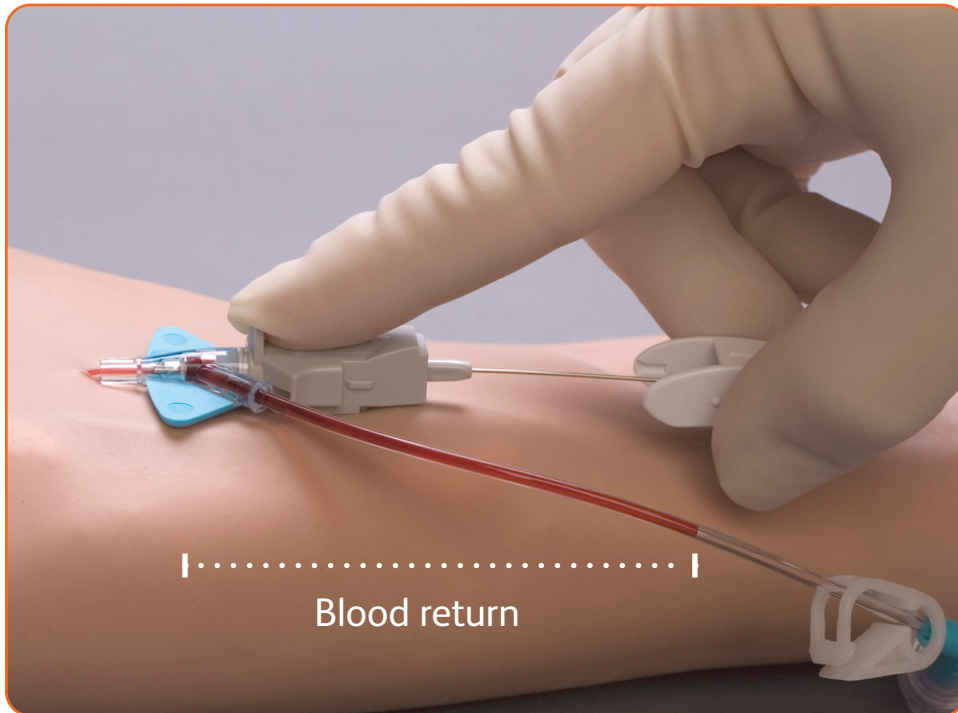


Secondary flash

- Blood return continues up the extension
- Continuous flow confirms the catheter tip is within the vessel lumen
- Quickens as needle is withdrawn


Stabilize catheter and remove needle

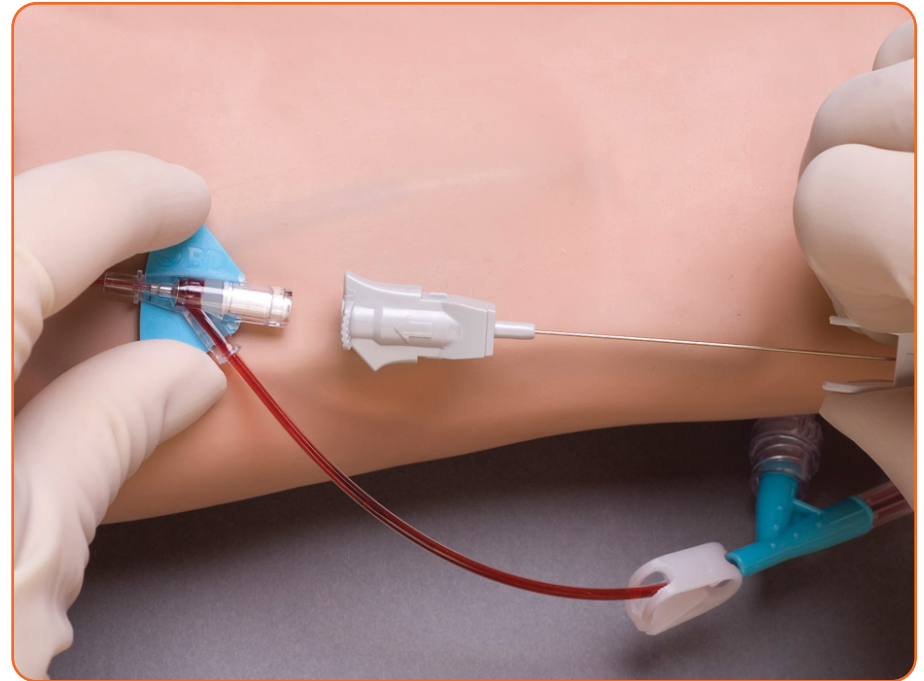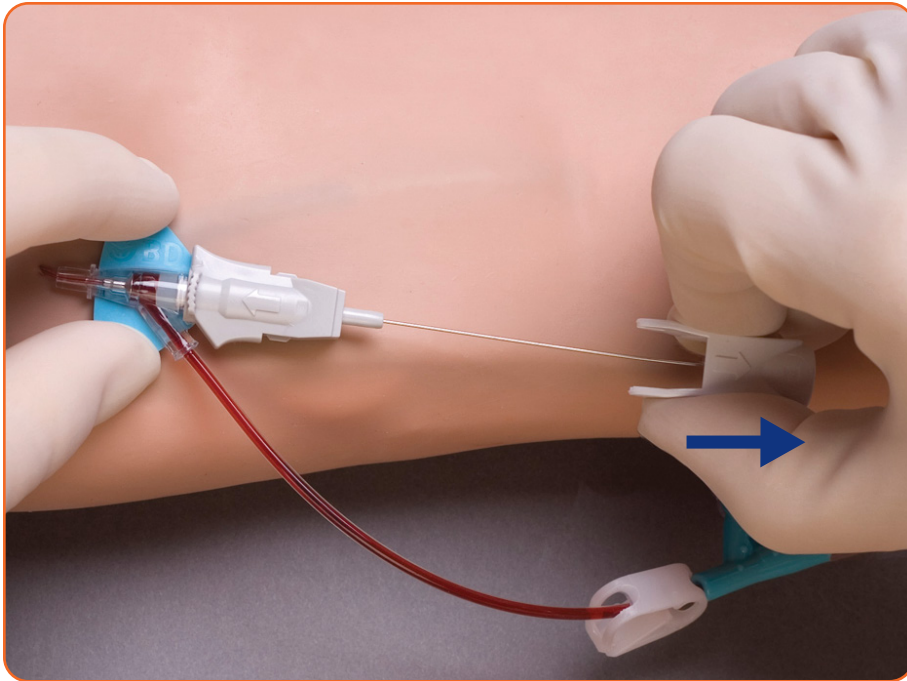

Nexiva™ Closed IV Catheter System—Dual Port

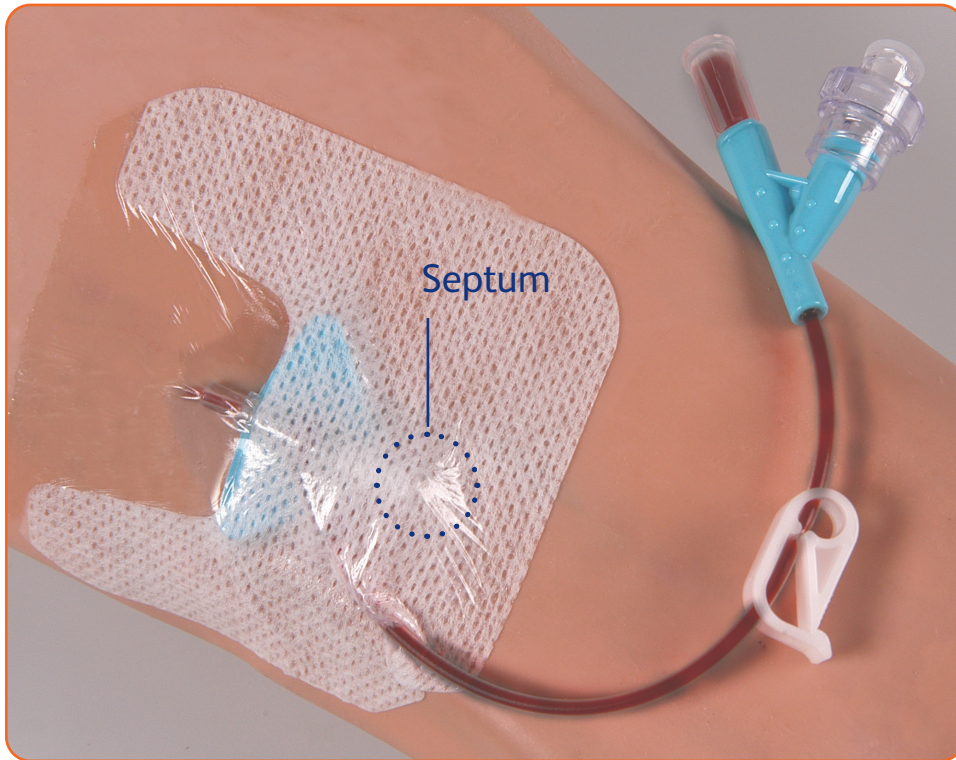

- Secure and apply the sterile dressing

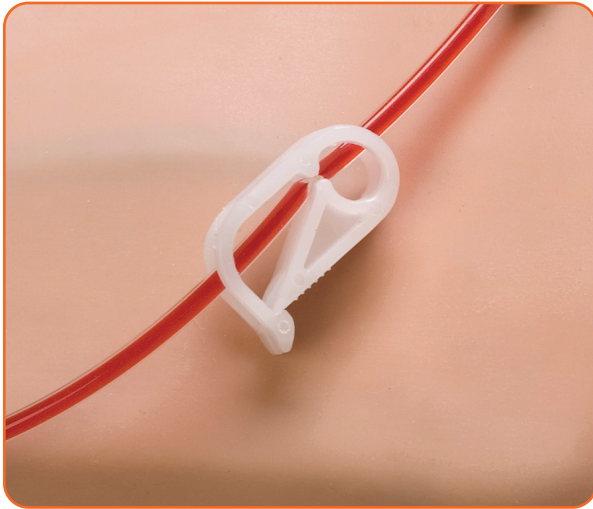

Clamp and replace vent plug


- Clamp extension set
 - Prevents blood leakage
- Replace vent plug with Q-Syte™ Device





Unclamp and flush

- Release the clamp to flush or infuse
- Complete securement of extension tubing
- Single lumen catheter
 - Compatible meds only

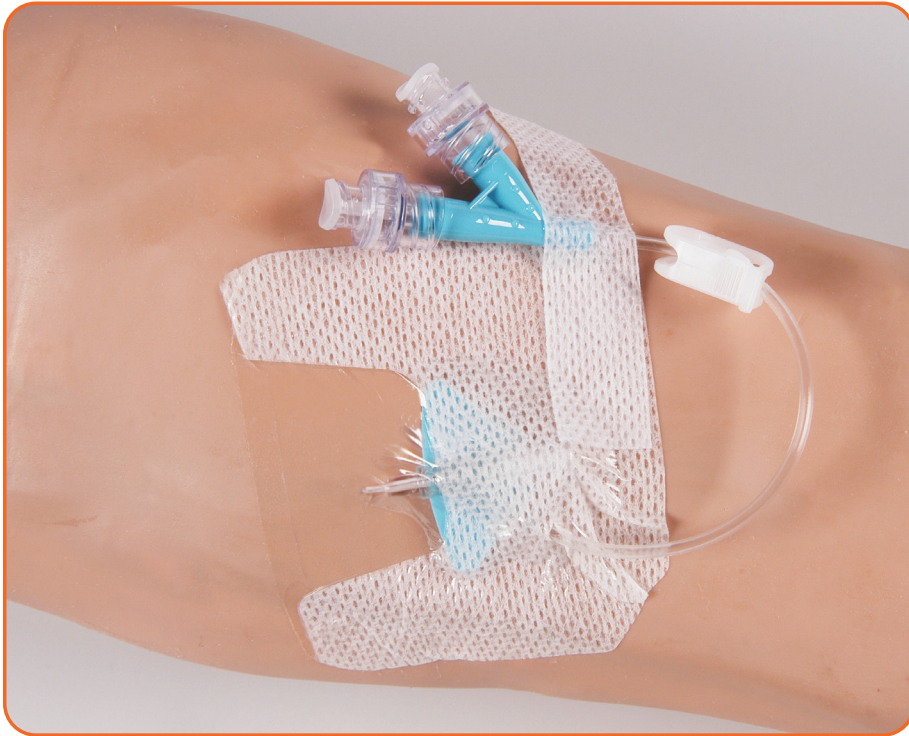

Disconnection:

- Clamp before disconnecting to minimize the reflux of blood
- Hold the Q-Syte™ Device during disconnection and use a straight-off approach

BD, Sandy, UT 84070, U.S.

1.888.237.2762

bd.com/nexiva

bd.com

BD, the BD Logo, Instaflash, Nexiva and Q-Syte are trademarks of Becton, Dickinson and Company or its affiliates.
© 2024 BD. All rights reserved. BD-6039 (10/24)

